VAGHEGGI  
PHYTOSMETICI

# VGH PHYTOTECH FACE PLUS

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**Rehydra**  
Moisturising Cleansing Milk

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Cleansing Toner

## Lime

Make-up remover micellair  
Cleanser Vitamin C

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**Delay Infinity**  
Cleansing Milk

**Intense**  
Complete Cleanser face and eyes

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	PROGRAMME	STAGE NO.	DESCRIPTION
1	OILY SKIN	1	INTENSE PEELING
		2	HYPER-SMOOTHING MASSAGE+DELIVERY
		3	VIBROMASSAGE STAGE
2	DEHYDRATED/DRY SKIN	1	SOFT PEELING
		2	HYPER-SMOOTHING MASSAGE+DELIVERY STAGE
		3	STIMULATING VIBROMASSAGE STAGE
3	SENSITIVE SKIN	1	MEDIUM PEELING
		2	HYPER-SMOOTHING MASSAGE+DELIVERY
		3	DRAINING VIBROMASSAGE
4	OXYGEN-DEPRIVED SKIN	1	INTENSE PEELING
		2	STIMULATING VIBROMASSAGE
		3	HYPER-SMOOTHING MASSAGE
5	OXYGENATING	1	INTENSE PEELING
		2	STIMULATING MASSAGE + DELIVERY
		3	DELIVERY
6	WRINKLES	1	MEDIUM PEELING
		2	HYPER-SMOOTHING MASSAGE+DELIVERY
		3	DELIVERY
7	SKIN DARK SPOTS	1	MEDIUM PEELING
		2	HYPER-SMOOTHING MASSAGE
		3	STIMULATING MASSAGE + DELIVERY
8	BIODYNAMIC FIRMING	1	MEDIUM PEELING
		2	HYPER-SMOOTHING MASSAGE
		3	STIMULATING MASSAGE + DELIVERY
11	SOFT PEELING		
12	MEDIUM PEELING		
13	INTENSE PEELING		
14	GALVANIC CURRENT DELIVERY		UTILISER SPECIFIC SERUM/PRODUCT

# ULTRASOUND 25 kHz: deep cleaning

## PHASE 1 – FOLLICULAR CLEANING/ PEELING

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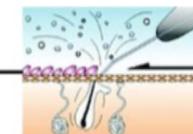
tip of the spatula facing down to facilitate exfoliation in phase 1 of peeling

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# PREVENTION AGE CARE LINES & FACE PLUS

	WHITE MOON	LIME
ÉTAPE 1	<b>BRUSHING</b> UTILISER: CLEANSING TONER	<b>BRUSHING</b> UTILISER: VITAMIN C MICELLAR CLEANSER
ÉTAPE 2	<b>ULTRASOUND SPATULA 25 KHz</b> UTILISER: FIZZY MASK & REHYDRA MOISTURISING TONING LOTION	<b>ELECTOPORATOR</b> UTILISER: ZINC+ORANGE MINERALISER & 2.5ML OF REHYDRA MOISTURISING TONING LOTION
ÉTAPE 3	<b>BIPOLAR RADIOFREQUENCY (30')</b> UTILISER: ECHINACEA FACE MASSAGE TECH	<b>ULTRASOUND SPATULA 25 KHz</b> UTILISER: BAMBOO EXFOLIATING GEL & ZINC+ORANGE
ÉTAPE 4	<b>ELECTOPORATOR</b> UTILISER SMOOTHING CONCENTRATED DROPS & 2.5ML OF REHYDRA MOISTURISING TONING LOTION	<b>BIPOLAR RADIOFREQUENCY (30')</b> UTILISER: ECHINACEA FACE MASSAGE TECH
ÉTAPE 5	BRIGHTENING MINERAL MASK	VITAMIN C FACE MASSAGE
ÉTAPE 6	PROTECTIVE BRIGHTENING EMULSION	VITAMIN C FACE MASK
		VITAMIN C FLUID OR VITAMIN C CREAM

# AGE CARE LINES & FACE PLUS

	DELAY INFINITY	INTENSE	75.15
ÉTAPE 1	<b>BRUSHING</b> UTILISER: CLEANSING MILK	<b>BRUSHING</b> UTILISER: COMPLETE CLEANSER FACE & EYES	<b>BRUSHING</b> UTILISER: CLEANSING CREAM
ÉTAPE 2	TONING LOTION	<b>ULTRASOUND SPATULA</b> 25 KHz UTILISER: EXFOLIATING GEL & REHYDRA MOISTURISING LOTION	<b>ELECTROPORATOR</b> UTILISER: PRECIOUS TONER
ÉTAPE 3	<b>ULTRASOUND SPATULA</b> 25 KHz UTILISER: BAMBOO EXFOLIATING GEL & TONING LOTION	<b>BIPOLAR RADIOFREQUENCY</b> UTILISER: ECHINACEA FACE MASSAGE TECH	<b>ULTRASOUND SPATULA</b> 25KHz UTILISER: FACE SCRUB & PRECIOUS TONER
ÉTAPE 4	<b>BIPOLAR RADIOFREQUENCY</b> UTILISER: ECHINACEA FACE MASSAGE TECH	DEEP FACIAL MASSAGE	<b>BIPOLAR RADIOFREQUENCY</b> UTILISER: ECHINACEA FACE MASSAGE TECH
ÉTAPE 5	FACE MASSAGE VIAL	<b>ELECTROPORATOR</b> UTILISER: SCHOCK CONCENTRATE & REHYDRA MOISTURISING LOTION	GEL LIKE MASK
ÉTAPE 6	FACE MASK	SENSITIVE ZONE MASK T ZONE MASK	INTENSIVE EYE CONTOUR
ÉTAPE 7	<b>ELECTOPORATOR</b> UTILISER: FLAWLESS FACE VIAL & 2.5ML OF TONING LOTION		SMOOTHING DAY CREAM
ÉTAPE 8	EYE AND LIP CONTOUR CREAM		
ÉTAPE 9	DAY CREAM		

## CE TECHNICAL CHARACTERISTICS

MANUFACTURER	TECHNOLOGY SRL VIA GARGANO N 7-B 20139 MILANO Tel. 0039 02 57303118 Fax. 0039 02 533174
MODEL: NAME:	VAG.1109 VGH FACE PLUS
POWER SUPPLY:	From single-phase 230 V $\square$ 10% 50/60Hz, via plug / socket with protective conductor.
Absorbed power:	170 VA
External protection:	 2 FUSES T 1,6 A -250 V
<b><u>OUTPUT POWER:</u></b>	
Scrub brushes	24 v.cc 1,3 a n° 315 RPM
Ultrasound 25khz	frequency 25 khz +/- 10% mechanical oscillation 30 micron pp +/- 20% power 9w
Electroporation (electro stimulation)	electrical pulse generator max.27ma current on 2 plates 0,3 ma/ cm <sup>2</sup>
Radio frequency	10 watt 450 khz $\pm$ 10 % 1hz -500 hz (square wave) 1hz -500 hz (square wave)
Ionophoresi (constant current)	24 v.cc max.4ma < 0,9 ma /cm <sup>2</sup> (square wave) on the 2 electrodes supplied
Ultrasound 25kHz	frequency 25 khz +/- 10% mechanical oscillation 30 micron pp +/- 20% power 9w
Classification:	CLASS I
Physiological effects:	SEE MANUAL

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#### ENVIRONMENTAL OPERATING CHARACTERISTICS:

environmental temperature:	$\pm +10^{\circ}\text{C}$ A $\pm +35^{\circ}\text{C}$
humidity:	from 30 a 75%
atmospheric pressure:	from 700 a 1060 hPa

#### TRASPORT AND STORAGE:

temperature:	$-10^{\circ}\text{C}$ A $\pm +50^{\circ}\text{C}$
humidity:	from 10 a 80%
atmospheric pressure:	from 600 a 1060 hPa
DIMENSIONS:	L 495 mm x P 495 mm x H 1008mm 45 Kg

NOTE: The manufacturer reserves the right to make alterations to the equipment described in this manual at any time and without prior notice.

The equipment has been programmed and projected to have results on aesthetic imperfections without overpass the allowed aesthetic limit

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